


Patterns

Turning data into discussion

RocheDiabetes Care Platform allows you to quickly and easily visualize your patients' self-monitored blood glucose data to discover patterns and draw attention to areas that might need intervention.

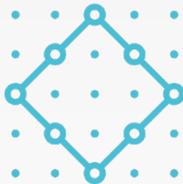

RDCP provides visual indicators for **up to 21 different glucose patterns** organized into five predefined pattern groups:

- **Hypoglycemia**
- **Hyperglycemia**
- **Variability**
- **Use of system**
- **Treatment adherence**

With Patterns, you can create a collaborative conversation around your patients' self-monitoring by acknowledging positive patterns, as well as discussing patterns outside desired thresholds, and engage in a discussion about causal factors and potential changes for therapy optimization.




Glucose measurements that trigger any enabled patterns are highlighted in red or blue in the graph, and group indicators are shown next to each pattern group.

With help from RDCP and Patterns, you can make consultations more efficient and help foster patient learning and engagement.

How to set up Patterns

On the RocheDiabetes Care Platform

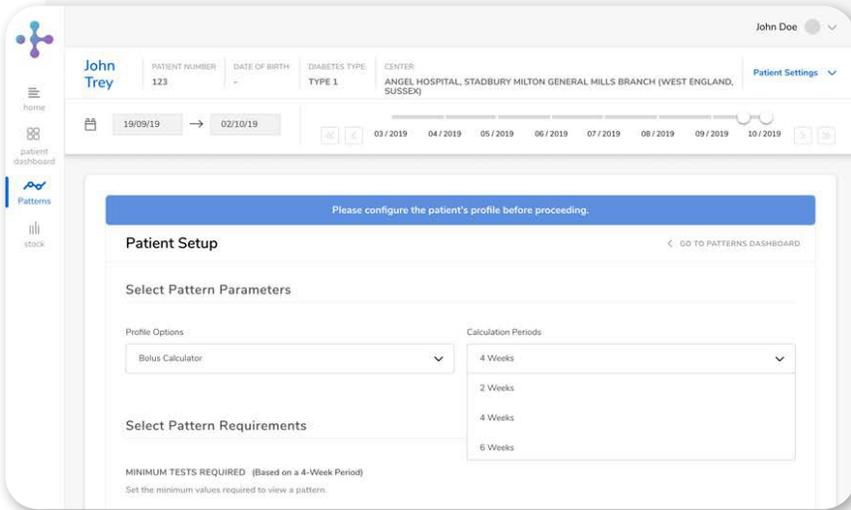

1 First, click on the **Patterns Icon** on the left vertical navigation panel on the **Patient Dashboard**.

If this is the first time you are accessing **Patterns**, you will be directed to the **Patient Setup** screen.

2 Configure the **Pattern Parameters** for your patient.


3 Review the **Minimum Tests Required** to view a pattern and the **Glucose Target Ranges**. These are initially predefined but can be customized for each patient.

4 The Configure Patterns section will allow you to select up to 21 different glucose patterns organized into five groups: **Hypoglycemia, Hyperglycemia, Variability, Use of System, and Treatment Adherence**.

5 In the **Configure Patterns** section, you can also customize exactly when RDCP will highlight a pattern.

6 After completing your **Patient Setup**, click **Save Configuration** and confirm.

7 Once completed and saved, the **Patterns Icon** will now open the **Patterns Dashboard** for your patient.

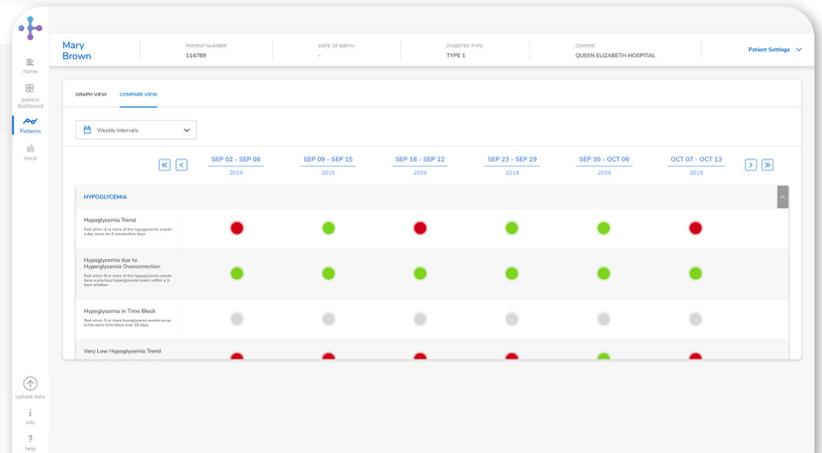

Remember, a minimum number of tests are required for results to be calculated, and data upload from a patient's device is required for results to display on the **Patterns Dashboard**.

Roche Diabetes Care is aware the platform may need to undergo a Data Privacy Impact Assessment. Please contact your Roche Sales Representative if you would like assistance during this process.

¹As stated in the terms and conditions of the RocheDiabetes Care Platform, Roche Diabetes Care will act as a data processor for the healthcare organization (data controller). To ensure compliance, Roche Diabetes Care created a Data Processing Agreement that stipulates how Roche Diabetes Care processes data on behalf of the data controller.

ACCU-CHEK, ROCHE DIABETES CARE, and ROCHEDIABETES CARE PLATFORM are trademarks of Roche. All other product names and trademarks are the property of their respective owners.

© 2020 Roche. 348-56685A-1020



Want more information?

diabetescareplatform.com | 1-800-628-3346
Or work with your sales rep directly.